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PRODUCED BY MML Molecular Packaging, Inc., Trousdale, OR 97060 USA	ABBOTT Abbott Molecular Inc., 5000 Glenborough Avenue, Des Plaines, IL 60018 USA	CE	PAP

EN	ES	FR

DA	SV	FI

NO	NL	CS

ET	PL	ZHHK

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You will receive a kit containing the following:
1 Tube (Sample Tube)
1 Swab

If you do not follow the instructions, your result may be incorrect.

Store kit before and after use at room temperature (between 15°C to 30°C).
Return your sample within 24 hours.
Do not use if you are pregnant.
Do not clean genital area immediately before collecting sample.

DO NOT touch the white tip of the swab or lay the swab down. If the white tip is touched or the swab is laid down or dropped, your results may not be accurate.

Contact your healthcare provider with questions regarding test results/ diagnosis or assistance with obtaining a new kit.

Additional product support please visit:
www.molecular.abbott/int/en/simpli-collect-hpv

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Find a comfortable position to easily access the opening of your vagina.
Hold swab with one hand. With the other hand, move the folds of skin (labia) out of the way to expose the opening of your vagina.

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EN SNAP DE ABBRECHEN FR CRAC
Hold tube upright. With the white tip pointed down, put the swab into the tube.
Snap the orange swab handle by bending the handle against the tube at the break line.

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Hold tube upright and recap the tube by screwing the cap clockwise.
When you feel resistance, keep turning about another quarter turn until cap no longer moves.

Hold tube upright and recap the tube by screwing the cap clockwise.
When you feel resistance, keep turning about another quarter turn until cap no longer moves.

Return your sample within 24 hours following the instructions provided separately.

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